

MATRIX ROTATIONAL LAL MATTRESS QUICK GUIDE



SETTING UP

1. Open the hooks on the back of the control unit and attach it to the footboard
2. Attach air mattress hoses at the foot of the bed to the control unit, and lock in place
3. Make sure the quick deflate bolster hose at the foot of the mattress is coupled with the quick coupling connector and **locked** in place
 - Make sure the hoses are secure by tugging both hoses firmly
 - Make sure all hoses are freely suspended without being pinched or kinked
4. Uncoil the power cord and plug into the correct AC power source
 - Make sure cord is not pinched
 - Make sure nothing is lying on top of the cord
 - Make sure the cord is safely out the way and not a tripping hazard

OPERATING THE UNIT

1. Power **ON** by pressing the Power button on the control unit
2. Press **MAX INFLATE** to rapidly air up the mattress quickly
3. Once inflated, set desired rotation
 - Press **SELECT** key to choose desired rotation angle (Left, Right, Center, etc.)
 - **UP/DOWN** keys set desired turn times and turn angles
 - Press **TURN** to activate rotation therapy
4. **COMFORT CONTROL** key sets patients' weight
 - Use **UP/DOWN** keys on COMFORT CONTROL setting to customize comfort level
 - UP makes the mattress more Firm
 - Down makes the mattress Softer
5. **LOCK** unit by pressing and holding the LOCK icon until the green light appears
6. **Unlock** the unit by pressing and holding the LOCK icon until the green light disappears
7. **PAUSE** is used to pause rotation therapy
8. An Audible & Visible **ALARM** will sound in the event of a power outage or if the hoses become disconnected from the unit.

IMPORTANT SAFETY INFORMATION!

CPR RELEASE: PRESS SIDES OF HOSE TUBING THAT CONNECTS TO BLOWER UNIT AND TUG FIRMLY TO DISCONNECT FOR RAPID DEFLATION

NEVER ELEVATE HEAD OF BED OVER 30 DEGREES WHILE PATIENT IS ROTATING

ROTATIONAL PRODUCTS DO NOT REPLACE REPOSITIONING PATIENTS AND TURN SCHEDULE PROTOCOLS

FOR FURTHER ASSISTANCE CALL US AT 1-866-350-5640 OR REACH OUT TO YOUR LOCAL CUSTOM MEDICAL SOLUTIONS OFFICE