


Patriot LAL-AP

Alternating Pressure Mattress System with Low Air Loss Vented Cover QUICK REFERENCE GUIDE

The Patriot LAL-AP is a mattress system that provides alternating pressure relief therapy with low air loss through the vented cover.

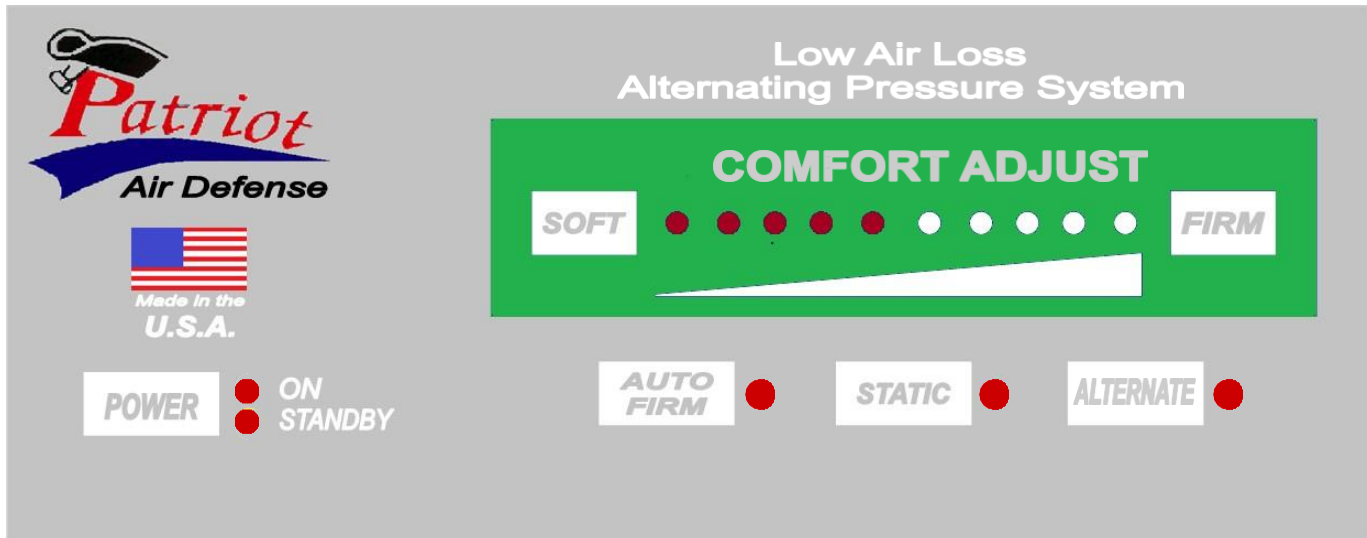

CONTROL UNIT OPERATION

POWER: When the unit is plugged in the STANDBY light will illuminate. Pressing the POWER button will illuminate the ON light and the unit will begin to inflate. The ON light must be on when the patient is on the mattress.

STATIC: The unit starts in the STATIC mode of operation. This mode delivers constant or static pressure.

ALTERNATE: In the ALTERNATE mode of operation the mattress will continuously alternate the pressure in adjacent cushions to provide alternating pressure relief.

AUTO FIRM: Pressing the AUTO FIRM button will provide maximum inflation for a period of 3 minutes providing a uniformly firm surface to assist in nursing care, repositioning and transfers. After 3 minutes the unit will automatically return to its previous comfort setting and mode of operation.

COMFORT ADJUST: Although the mattress is providing continuous alternating pressure relief the overall firmness of the mattress can be adjusted using the COMFORT ADJUST control.

CPR: To quickly deflate the mattress remove the hoses from the control unit. This will rapidly deflate the mattress.

KEYPAD LOCKOUT: Hold down the hidden key under the Flag for 3-5 seconds, until the light next to the flag illuminates. This will lock-out the unit to prevent unauthorized adjustments. Press and hold the Flag again until the light goes out to unlock the keypad.

POWER LOSS PROTECTION: The LAL-AP will stay inflated during temporary power interruptions or outages.